HEALTH CARE

Lighting and
Bedhead Units

atena *lux*[®]

INDICE / *INDEX*

- 2 AZIENDA / *COMPANY*
- 6 HUMAN CENTRIC LIGHTING
- 8 CASO STUDIO / *CASE STUDY*

12 TRAVI TESTALETTO OSPEDALIERE *BEDHEAD UNITS FOR HOSPITALS*

14 NORMATIVE *REGULATIONS*

15 PRETUBAGGIO *PRE-PIPING*

- 18 AUXILIA

- 24 SENSE

- 28 SANIMED
SANITECH SYSTEM

- 34 ARMONIA

- 40 THERAPY

- 54 TRIAL

**60 TRAVI TESTALETTO
PER RESIDENZE SANITARIE ASSISTITE
*BEDHEAD UNITS FOR NURSING HOMES***

● **62 AIRWALL**

● **66 STAIR CARE**

● **68 SKYWALL**

● **70 WAY WALL**

● **72 LEVEL**

● **74 HORIZON**

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**78 FINITURE E COLORE
*FINISHINGS AND COLOURS***

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**84 SISTEMA CERTIFICATO
PER SALE OPERATORIE
*CERTIFIED SYSTEM
FOR OPERATING THEATRES***

**85 TAURUS: ILLUMINAZIONE PER SALE
OPERATORIE
*TAURUS: LIGHTING FOR OPERATING
THEATRES***

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Founded in 1994, ATENA LUX is a dynamic and young company, recognised as manufacturer of lighting solutions and bedhead units on domestic and foreign markets. The production area, located in Gruaro (VE), covers a surface of 4933 square meters and boasts several latest technology machines. From design to production, a highly qualified staff creates customized quality solutions that are synonymous with guarantee, comfort and safety. Thanks to profuse research efforts, in these last years the company has grown considerably in the medical sector, increasing its offer with new models of bedhead units and specific lamps for operating theatres.

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CUSTOMER CARE

A constant communication with the customer has allowed the company to transform and shape its creations according to the specific customer's needs. The added value, that distinguishes the ATENA LUX products, is the company's ability to listen to specific exigencies and producing customized products. Thanks to several services that the company provides, like consulting, project development and lighting planning calculation, ATENA LUX has established a continuous relationship based on trust with customers.

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PRODUCTION PROCESS MADE IN ITALY

Over the years the production sector has evolved at the same speed with the evolution of the raw materials and the processing technologies. Taking inspiration from the Lean Manufacturing logic, the efficiency of production processes and the adoption of new production methods have allowed the company to maintain a competitive quality and economic standard on the market. Products take shape through laser cutting, punching and processing of aluminum extrusions. At the same time, keeping the assembly and wiring process manual, it was also possible to maintain the flexibility that is often required for site exigencies. Thanks to the presence of an internal photometric laboratory, the lighting fittings are subjected to lighting tests that enable to obtain the necessary data for lighting project elaborations.

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THE COMPANY'S QUALITY PRINCIPLES

The production system follows a qualitative process for the certification of the electromedical devices. The products, whose warranty can be extended up to 5 years, are subject to final testing before being put on the market, through specific tests in line with the requirements of the reference normative. The scrupulous analysis on the choice of raw materials and the concrete and fair collaboration with the suppliers remain the starting point for the development of reliable and safe products.

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SOCIAL RESPONSABILITY

The adoption of a development strategy based on sustainability issues is the basis of a long-term process that ATENA LUX has decided to start voluntarily. The company is approaching the issues of sustainability starting from the positive values of the management. Aware that the innovation processes that come to life in the company are reflected on society through economic strategies, social and cultural projects, ATENA LUX is projected towards a transformation and growth that involves not only the company but also the society.

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"light is a powerful non-pharmacological instrument that can help people in different ways to feel better"
LRC Lighting Research Center

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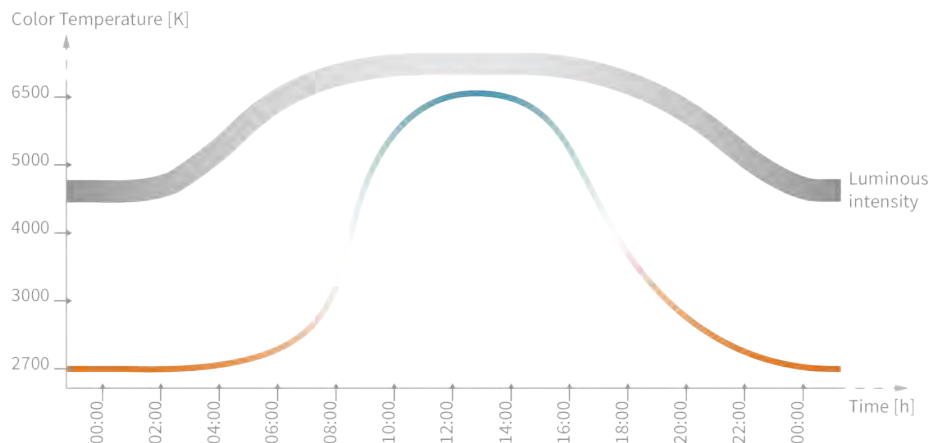
When light can make hospital environment more comfortable

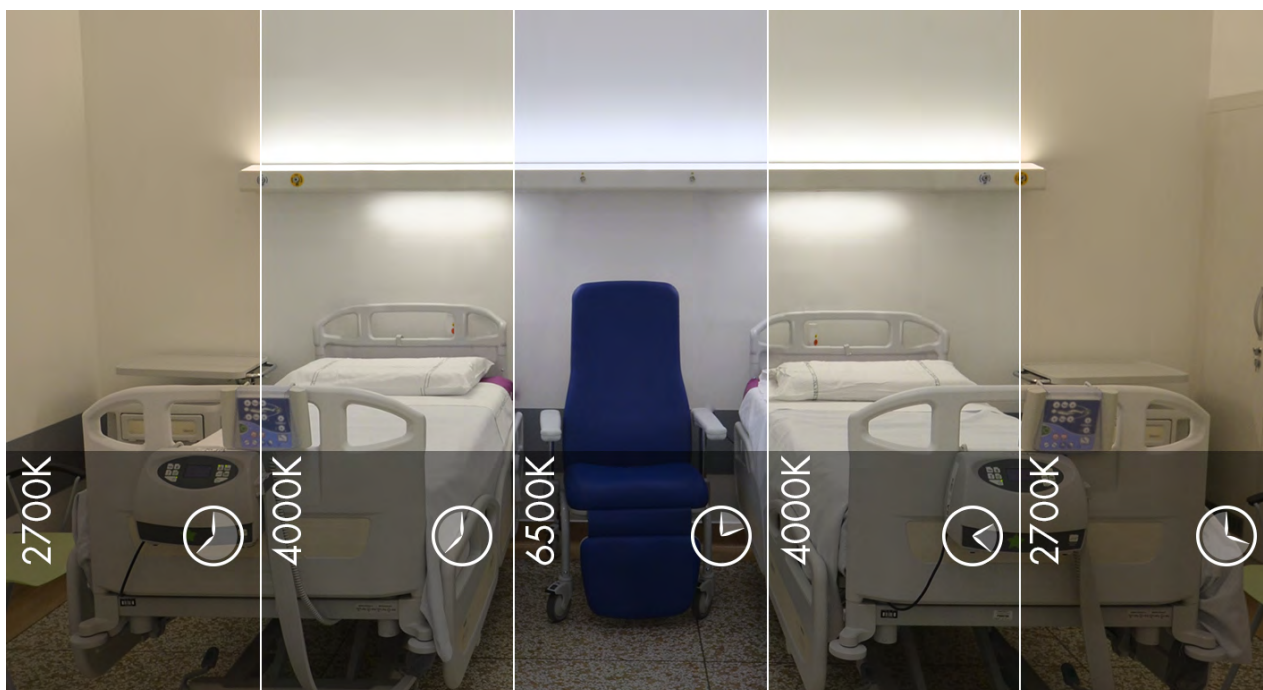
HCL (Human Centric Light) principle is very well appreciated inside hospitals and recovery clinics. In these places the natural light is almost absent and it is necessary an artificial light that respects the human natural biorhythms. The HCL systems bring visible advantages to the quality of patient's stay, helping the rehabilitation and a comfortable recovery.

ATENA LUX bedhead units and lighting fittings integrate LEDs with different colour temperatures and dimmable drivers. The lighting management is realized through an automatic regulation system synchronized with an astronomical clock at local time.

ATENA LUX lighting fittings are equipped with LED modules with different colour temperature (2700K- 6500K). According to its specific

exigencies, the company has realized some linear LED modules that fit perfectly to the existing lighting bodies. HCL system includes a central management unit that is programmed according the geographic position or a particular exigency. It also includes a real-time clock which allows for the automatic progression of the lighting intensity and colour temperature according the natural change of the circadian rhythm. In combination with a lighting and movement sensor, additional energy savings can be achieved. Controlling this system is easily possible via push-button or Touch panel.

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HCL SYSTEMS FOR PATIENT'S ROOMS

Patients who spend most of the day indoors, due to illness or mobility problems, experience an alteration of the sleep-wake rhythm, in particular, patients afflicted with dementia and other cognitive diseases suffer the lack of natural light.

In the hospital rooms, we try to faithfully reproduce the natural cycle of the light, replicating the same colour and intensity of light at dawn, in the afternoon and at sunset. The effects in patients and medical staff are positive and different. For patients, artificial light can compensate for the lack of natural light and restore a good sleep-awake rhythm, reducing the stay in the structure and the use of antidepressants.

Benefits are relevant also for medical staff: HCL systems prevent the onset of insomnia, increase physical well-being and improve the relationship with patients, moreover they reduce headaches and tired eyes in free time.

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HCL SYSTEM FOR INTENSIVE CARE UNITS

Lighting plays a key role in intensive care units. It must satisfy the complex clinical needs of the department and at the same time respect the state of mind of patients and their families, mitigating anxiety and disorientation.

Lighting systems must be designed in order to enable the medical and paramedical staff to carry out their activities properly, observing the patients and the equipment. At the same time, lighting must have a pleasant and comforting effect.

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CASE STUDY:

AMBIENT LIGHTING IN A PATIENT ROOM

Enlightening hospital premises, we have to consider that the patient usually requires a comfortable, diffused light with low intensity and without direct glares.

In order to take care of the patient and to ensure that visiting and treating take place in the best environmental conditions, it is necessary to have a direct lighting on the whole surface of the bed, in order to guarantee proper lighting levels for the visual tasks that have to be carried out by the hospital staff.

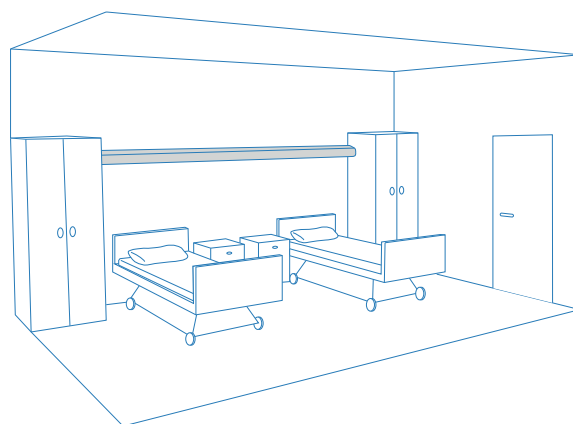
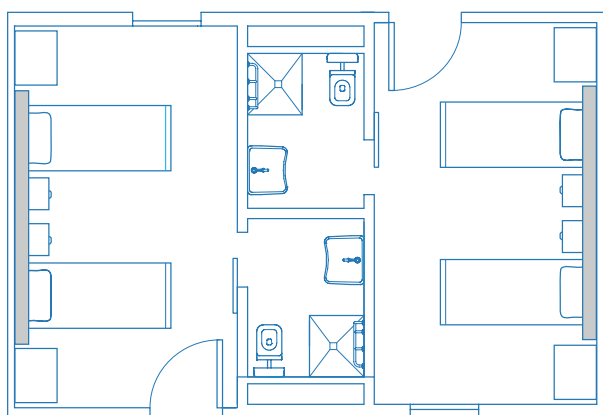
Moreover, the need of relaxation of the patient that has to stay in bed for a long time needs to be considered. This is the reason why the patient should have at disposal a light, which allows reading without disturbing other people in the room.

Light in patient rooms has to answer to several general requirements. Lighting levels must change according to the occurred activity, light can be extremely low (5 lux for night light), medium level for the general lighting (100 lux) or very high (1000 lux) for an accurate medical examination necessary for the patient.

EN 12464-1 and **DIN 5035** regulations recommend **minimum lighting levels** in the patient room necessary for the medical staff and for the patient activities. Thanks to the use of LED sources and the reduction of installation and maintenance costs of the electrical system, ATENA LUX bedhead units allow to obtain a great **economic saving**. As you can see on the following project simulation, according with the referring regulation, to obtain a correct lighting of the patient room it is not necessary to install additional lighting fittings that sometimes can generate direct and disturbing lighting.

The ATENA LUX technical department is available to develop customized **lighting projects**.

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Pareti
Wall **60% + 6% (finestra/*window*)**

Soffitto
Ceiling **80%**

Arredi (chiari)
*Furniture
(light colours)* **70%**

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GENERAL LIGHTING

According to EN 12464-1 and DIN 5035 regulations, the necessary quantity of light to carry out normal activities (for patients and medical staff) is 100 lux calculated on the floor level.

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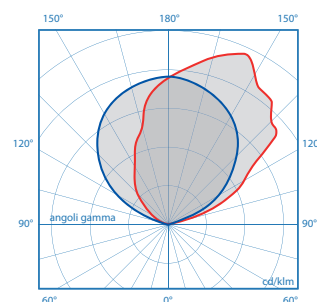
READING LIGHT

EN 12464-1 and DIN 5035 rules recommend a lighting level of 300 lux on a reading plane (dimensions: 90cm x 3cm, located at a distance of 1m from the head of the bed, 1,1m high and with an inclination of 75°).

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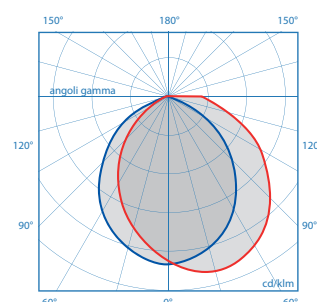
SIMPLE EXAMINATION LIGHTING

EN 12464-1 and DIN 5035 regulations recommend a minimum lighting level of 300 lux on the entire size of the bed, at 0,85m from the floor.

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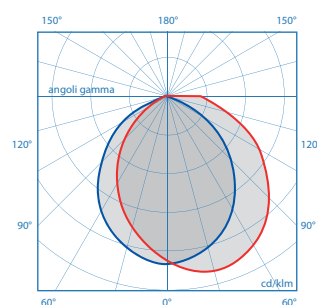
NIGHT LIGHT

Necessary quantity of light to guarantee the orientation of patients and medical staff during the night. It is given by an indirect emission through LED lighting source that must guarantee 5lux minimum at a 0,85m height.

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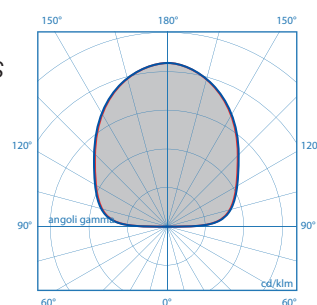
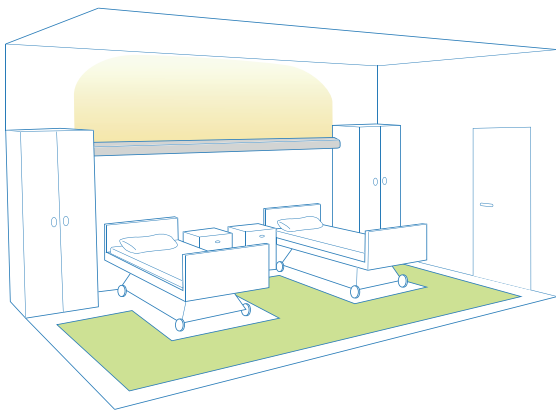
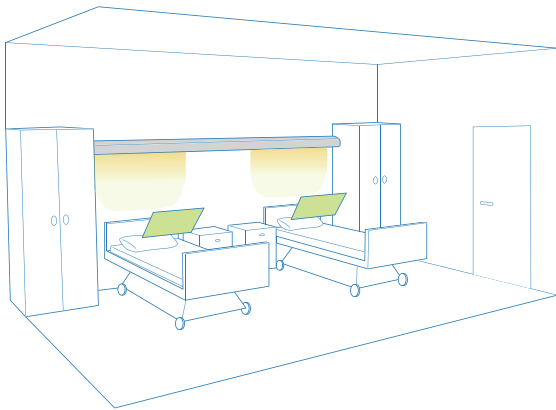
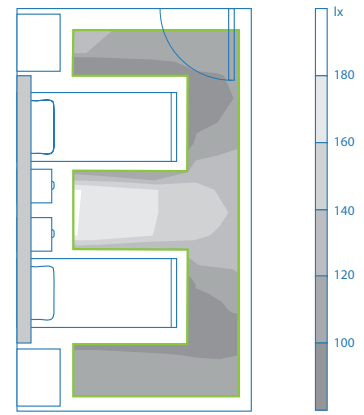
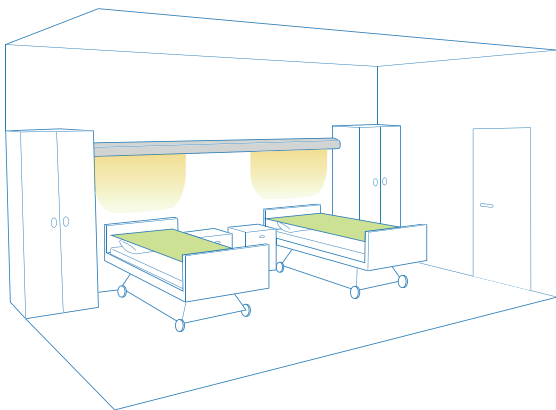
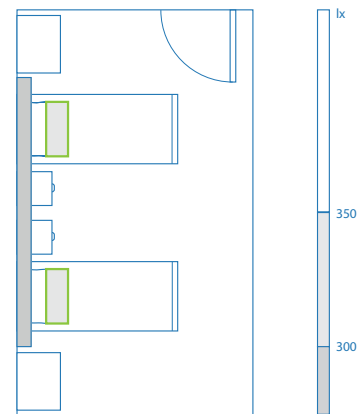
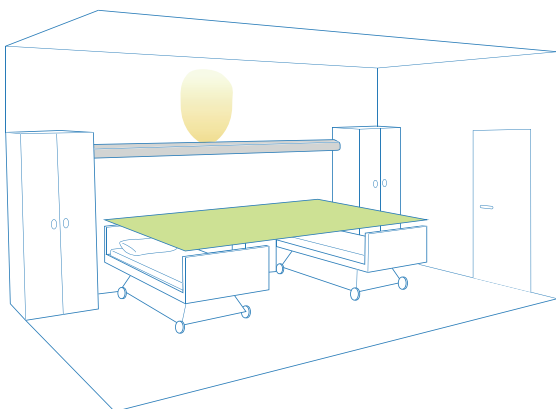
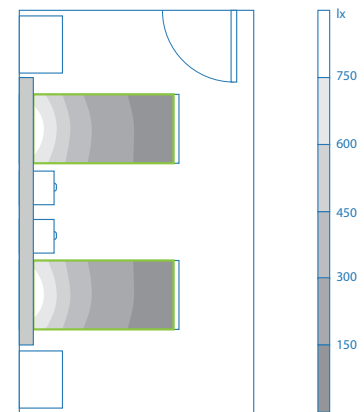
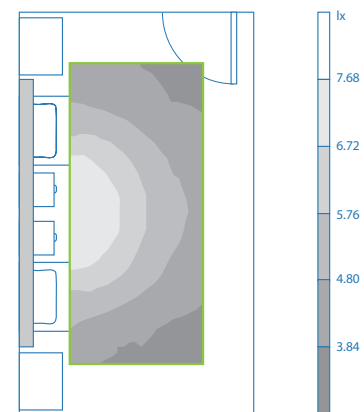
REGULATIONS

CERTIFICATION AND SECURITY

*ATENA LUX electromedical devices are compliant with all the European regulations regarding addition, they have the **CE branding** which is necessary and mandatory for all the electro-medical devices.*

*According to the **93/42/CEE directive** and further amendments, ATENA LUX bedhead units are **medical devices** for the installation in patient rooms, pre- and post-operating rooms, delivery rooms, physiotherapy rooms, clinics, laboratories.*

*The **UNI EN ISO 11197** and **CEI EN 60601-1** technical norms establish the suitability of bedhead units to be medical devices once examined their general characteristics, security and integrity.*

*According the **directive 93/42/CEE**, ATENA LUX bedhead units are medical devices belonging to class I, IIa, IIb. The bedheads units without predisposition for medical gases are included in class I.*

ATENA LUX bedhead units of class II are suitable for the installation or implementation of medical gases and anesthetics.

*Versions on which oxygen gas is not provided are classified on class IIa, whereas in case of the presence of oxygen gas they are part of class IIb. Bedhead units of class IIa and IIb are certified from the **ITALCERT** national authority with reference to the **074-02-02-DM certificate number**.*

- 93/42/CEE

European Directive about medical devices

- CEI EN 60601-1

Electro-medical devices

- UNI EN ISO 11197

Medical supply units

- EN 60598

Lighting devices

- EN 62471

Photobiological safety

- EN 55015, EN 61000

Electromagnetic compatibility

- UNI CEI EN ISO 14971

Medical devices

Application of risk management to medical devices

- UNI CEI EN ISO 13485

Medical devices

Quality management systems

Requirements for regulatory purposes

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EN ISO 13585

PRE-PIPING

*In order to enable the implementation of medical gases, ATENA LUX bedhead units of class IIa and IIb are equipped with houses **circuit and connections for technical gases** and allow the installation until four copper pipes for oxygen, protoxide, vacuum and compressed air distribution. In some specific situation, on request, it is possible to provide the **pre-piping**, that is the insertion and welding of the copper pipes to the gas outlets inside the bedhead units. This activity takes place directly inside the production plant during the bedhead units production process. The pre-piping activity is performed by **certified operators** who certify the suitability of ATENA LUX medical devices for the installation and testing of medical gas systems.*

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BEDHEAD UNITS FOR MEDICAL PREMISES OF GROUP 0, 1 AND 2

The electrical systems inside medical premises must comply with the instructions dictated by the CEI 64-8 standard. This standard defines the premises for medical use and their classification in 3 groups.

The classification of the medical premises occurs after a careful identification by the clinical engineering of the risks for the patient, and considers a specific knowledge of the electrical systems and electromedical devices inside the hospital.

The ATENA LUX bedhead units are considered medical supply units which integrate into a single device several services: electrical lighting, energy distribution, communication services, data transmission, therapeutic oxygen supply, medical gases, vacuum, sterile complex air, etc..

They are divided into these following groups:

GROUP 0

Medical premises where electromedical devices are not used directly on applied parts, therefore they do not come into physical contact with the patient.

Specific wall lighting fittings for nursing homes are part of this group: AIRWALL, STAIR, SKYWALL, HORIZON, WAY WALL, LEVEL.

GROUP 1

Medical premises where the applied parts are intended to be used externally or invasively in any part of the body except for the cardiac zone.

ATENA LUX bedheads that can be configured according to group 1 premises are: AIRWALL MED, AIRWALL CALL MED, AUXILIA, SENSE, SANIMED, ARMONIA, THERAPY, TRIAL.

GROUP 2

Medical premises where the applied parts are intended to be used for surgical operations or where the patient is subjected to vital treatment and a lack of feeding can be life threatening.

*According to ATENA LUX product development process, the bedhead units intended for medical premises of group 2 are made complete with:
protection of every single electric socket;
FM circuit divided into 2 circuits;
Network presence LED for each electrical socket.*

*This group includes all the hospital bedhead units (AUXILIA, SENSE, SANIMED, ARMONIA, THERAPY, TRIAL), whose configuration contains the **suffix .2**.*

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*Thanks to an accurate study of the technical and aesthetic details, AUXILIA combines functionality, assistance, cleanliness and elegance contributing to make the patient stay more comfortable and reliable. AUXILIA offers **direct, indirect and night light** and it is suitable for the installation of the **medical gases systems**.*

An appropriate bedhead unit for small and big hospital environments.

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Auxilia bedhead unit with channel for light distribution, channel for medical gases and UNEL sockets, data socket, equipotential sockets and LEDs for net connection. Can be also for suspended application, two columns are applied and its length is defined according with the system requirements (maximum length 2000mm each).

Columns are made with a steel C shaped profile and a steel covering. In every single column are located the connections of the electrical system or the connections of the medical gases. All this strong structure's sustained by a ceiling steel fixing plate.

The system must be anchored directly to ceilings (brick, reinforced concrete) able to support the weight of the bedhead unit and the suspensions, for this reason in addition the columns are supplied complete with a cover to hide the cutting of the false ceiling. The maximum weight of the bedhead unit per meter is about 20 kg and the columns weigh about 15kg per meter.

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*One channel bedhead unit realized with a compact design able to integrate all the utilities and accessories inside. SENSE is composed by a **minimal and elegant profile** assuring functionality to the medical staff during their routine activities and, at the same time, taking care of the patients comfort so that they can feel independent and relaxed during their stay.*

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*Designed to be a completely customizable system, the **vertical bedhead unit** SANIMED can combine the wall-mounted lighting fittings. This system combines efficiency and functionality all the main medical services necessary to medical staff and the indispensable functions for the patient's comfort. The possibility of having customized finishings makes SANITECH SYSTEM a modern product suitable for a perfect integration with the patient room, giving color and style to hospital places.*

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Supporting structure made of steel and frontal panel made of MDF

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IP20 *general*
IP40 *lighting circuit*
IP21 *electrical sockets*

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*Anodization,
antimicrobial coating,
decorative finishing*

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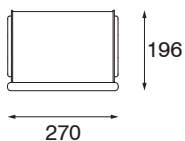
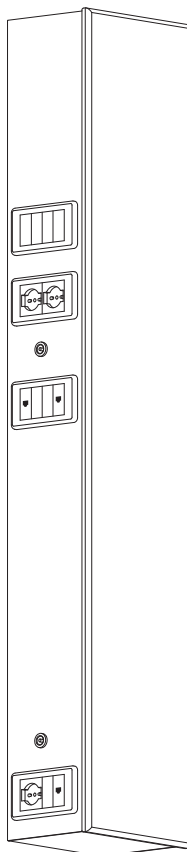
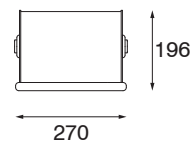
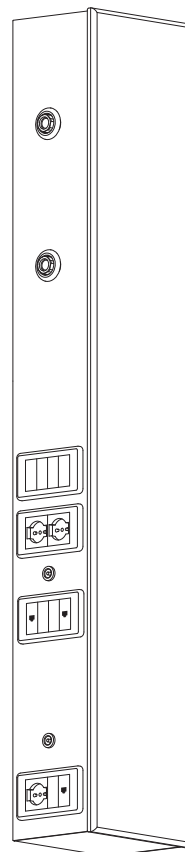
ELECTROMEDICAL SYSTEMS: SANITECH SYSTEM

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*ATENA LUX realize custom-made electromedical systems when the customer has a **specific technical and architectural requirement**. The project is developed according to the guidelines dictated by the reference normative and gives life to a **certified system**. The elements composing this system are realized with particular attention to aesthetics, ergonomics and patient well-being.*

The SANIMED vertical bed head unit, whose outlets are located on one side or on both sides, can be combined with a custom-made lighting applique. In order to remaining in line with the architectural standards required by the client, bedhead unit and lights are integrated using different materials, such as MDF wood, which offers a wide range of solutions and finishes. The lighting fitting is designed in such a way as to diffuse and indirect general light creating a relaxing atmosphere, an individual light for reading and for medical examination and night light. The MDF wood panel has the same finishing of the bedhead unit decorative covering.

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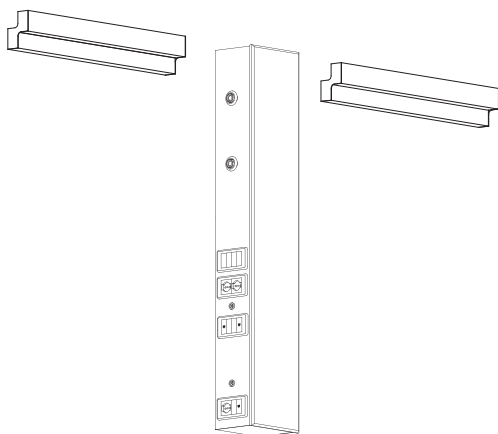
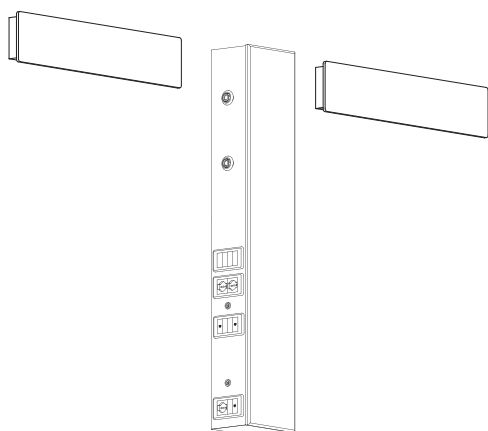
While the vertical bedhead unit meets the needs of electrical distribution and medical gases, the AIRWALL lighting fixture, with its soft and innovative design, maintains a light condition that is comfortable for the patient and at the same time suitable for carrying on the activities of the hospital staff. Thanks to its compact body, AIRWALL can be equipped with a low intensity general light, a no-glare reading light, night light and examination light.

SANIMED + STAIR CARE SANIMED + STAIR CARE

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The combination of the vertical bedhead unit with the STAIR PLUS lighting applique represents a more linear and technical solution. This lamp, in which direct light and indirect light are managed separately, can also be equipped with a hanging keyboard for the nurse call and also the management of direct light. Thanks to its refined and modern design, it is the perfect combination of its excellent lighting performance to the aesthetic and architectural needs of the environment.

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Thanks to its creative and decorative flexibility, WAY WALL enables to personalize the patient's room with style. Made of a double extruded aluminum profile, with reduced dimensions, can be personalized applying on the surface a decorative panel made of MDF wood, giving character to the environment. A simple and fresh solution that guarantees a soft light able to meet the lighting requirements of the patient room.

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*For more information about our lighting solutions, please read from page 60

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*ARMONIA is a **multifunctional panel** realized as a valid medical lighting element with the features of a furnishing accessory, with custom design and elegant finishing. ARMONIA guarantees the perfect lighting level inside the room, and thanks to the implementation of the medical gases systems, it becomes a perfect solution for those who want to renovate the hospital environment.*

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*ARMONIA has the **reading** and the **examination light**. The examination light needs the complete power of the LEDs while the reading light is settled by an electronic switching system that reduces the power and, as a consequence, the luminous flux of LEDs. The indirect general light is controlled separately and enables to completely illuminate the patient room.*

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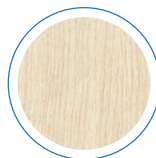
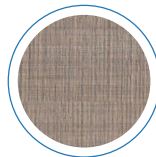
*ARMONIA is made of a painted steel structure and frontal panels made of **MDF** material, removable to guarantee internal inspection. These panels are **fire resistant** certified and **waterproof** and have the function of masking the wiring and the aesthetical function of characterizing the medical environment. On request, in addition to the traditional RAL colors, other wood finishing are available.*

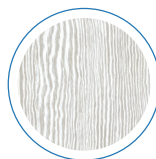
Thanks to the particular materials used for its production, ARMONIA results a long lasting product, suitable for the installation in places where hygiene and practicality are the main exigencies.

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Dimensional characteristics relating to one bed bedhead unit 1400 mm long.

ARMONIA devices can be custom-made according to technical exigencies.

For specific needs, please contact our Sales Department.

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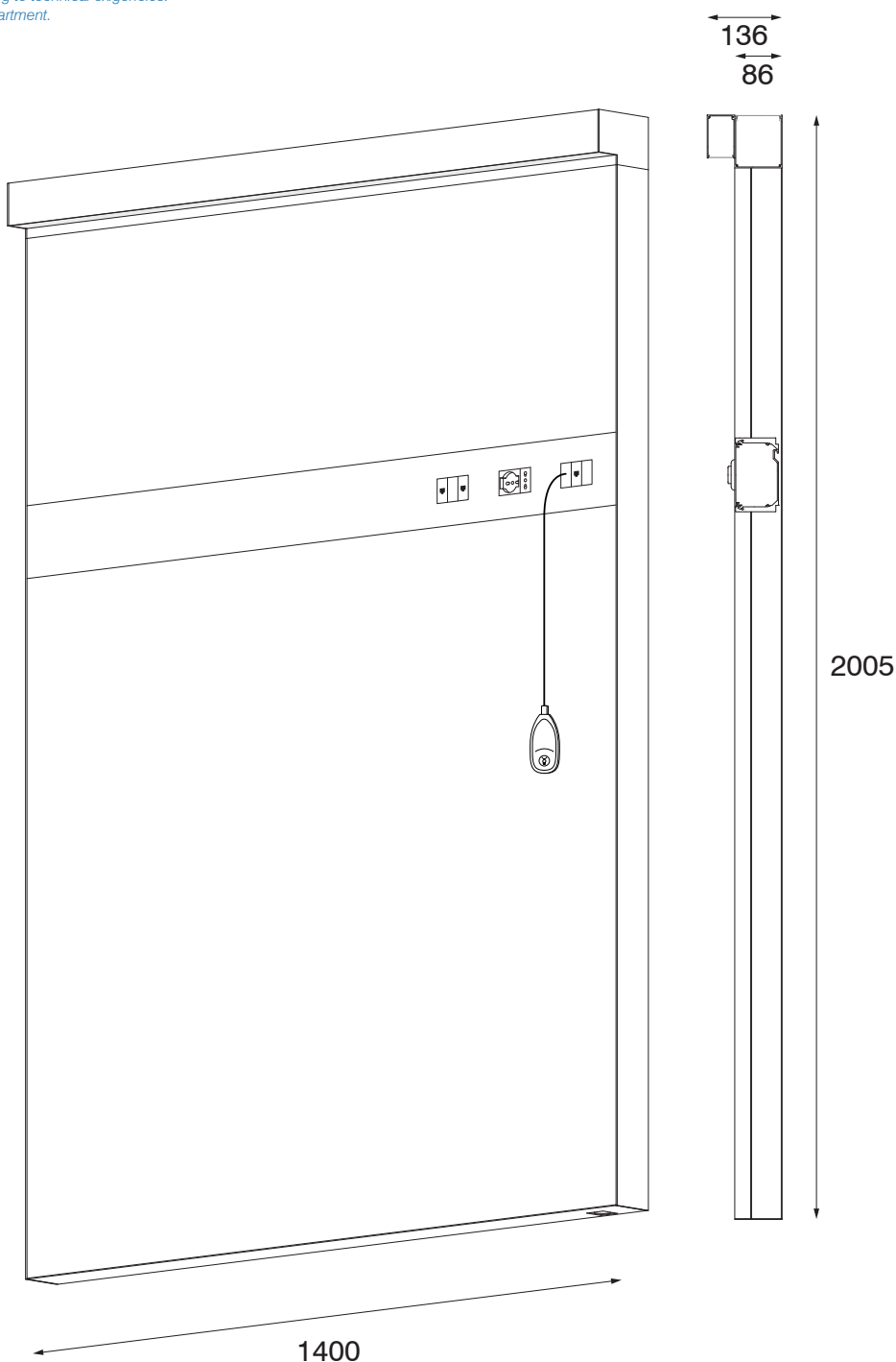
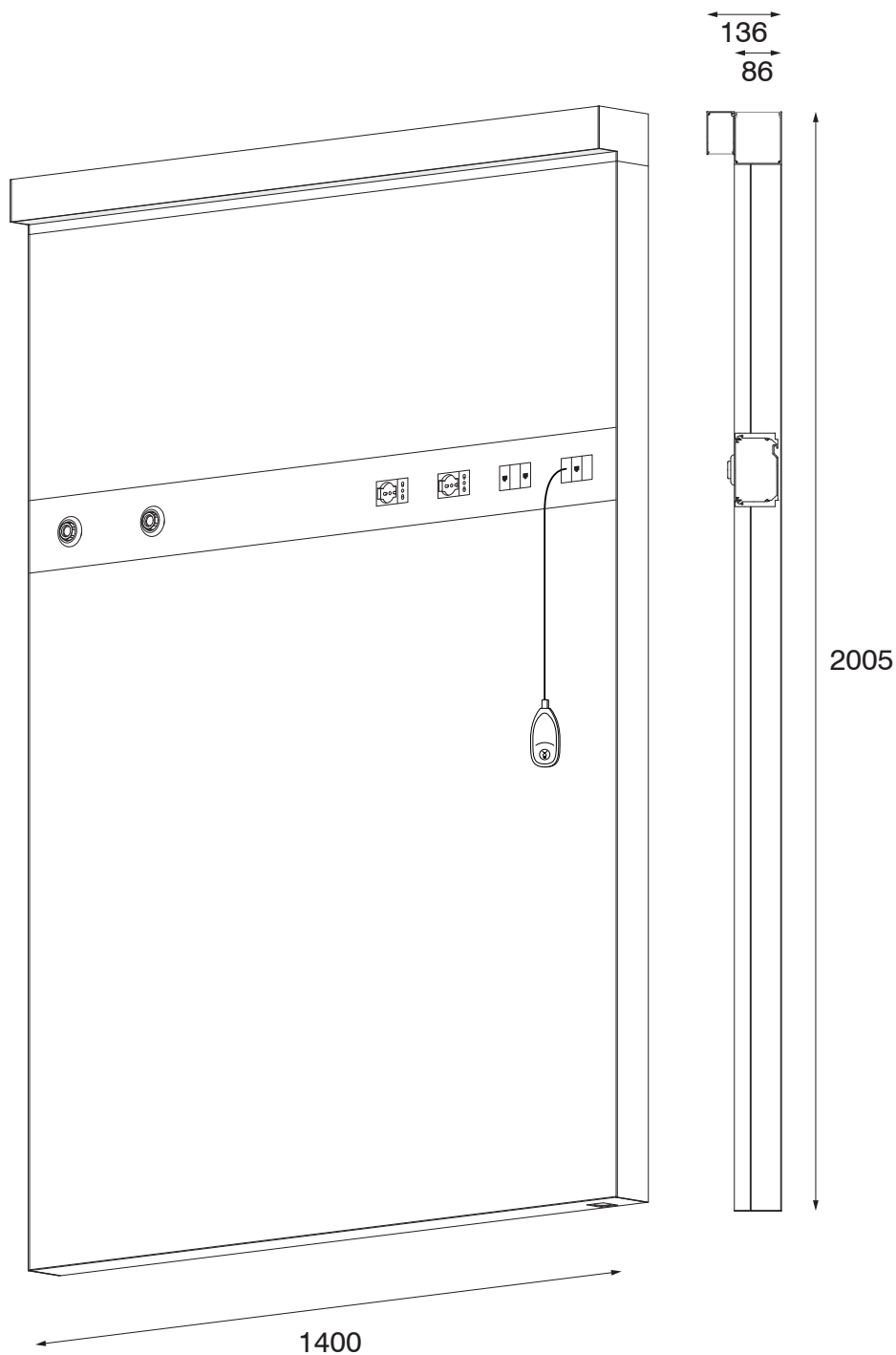
*THERAPY gets the most from space and allows a tidy disposition of the medical equipment around the bed, supplying **medical gases** and **electricity**. On request, it is possible to further integrate it. This device is suitable for the installation in medical places of the **group 2** such as anesthesia room, surgery room, intensive care, angiography and hemodynamic examination rooms.*

- › **THERAPY/W**
- › **THERAPY/S**
- › **THERAPY/V**
- › **THERAPY/R**

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*Medical device made for **medical premises of group 2** such as anesthesia room, surgery room, intensive care, angiography and hemodynamic examinations rooms. TRIAL is a modular solution composed of a maximum of three aluminum profiles in which electric utilities, medical gas system and call system can find an orderly and structured location. The version with lighting system has two additional channels for a comfortable light that efficiently supports the activity of the medical staff.*

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*These following versions are illustrative examples. TRIAL devices can be custom-made according to technical exigencies.
For specific needs, please contact our Sales Department.*

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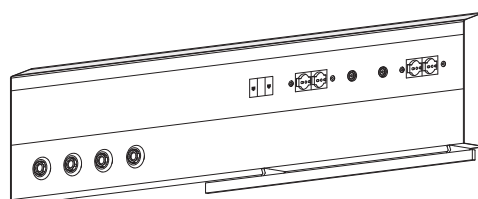
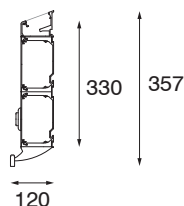
Thanks to its simple and elegant shape, AIRWALL fits perfectly in nursing homes, rest house, and rehabilitation clinics by making more modern and familiar spaces. Its light gives optimal visual conditions and patient wellness as well as a comfortable enlightening. Moreover, it allows the medical staff performing without difficulties.

- › **AIRWALL**
- › **AIRWALL MED**
- › **AIRWALL CALL**
- › **AIRWALL CALL MED**

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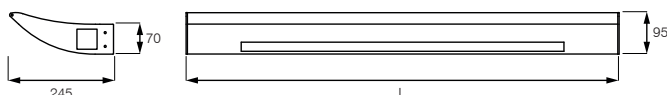
*In nursing homes and, in particular, in facilities dedicated to treating Alzheimer's disease, it is necessary to have **biologically effective artificial lighting**, suitable for daily activities and aimed at improving the quality of the sleep-wake rhythm.*

*In patients with **dementia**, who suffer more easily from sleep disorders, the application of the HCL principle helps to restore a regular circadian rhythm as much as possible.*

Thanks to a lighting system that follows the light of day, with variations in light intensity and color temperature, the positive effects are noted on the psychological and emotional state of the patients, but also on the assistants.

Potenza / **Power**

20 W

13 W

Temp. di colore / **Colour Temp.**

2700K+6500K

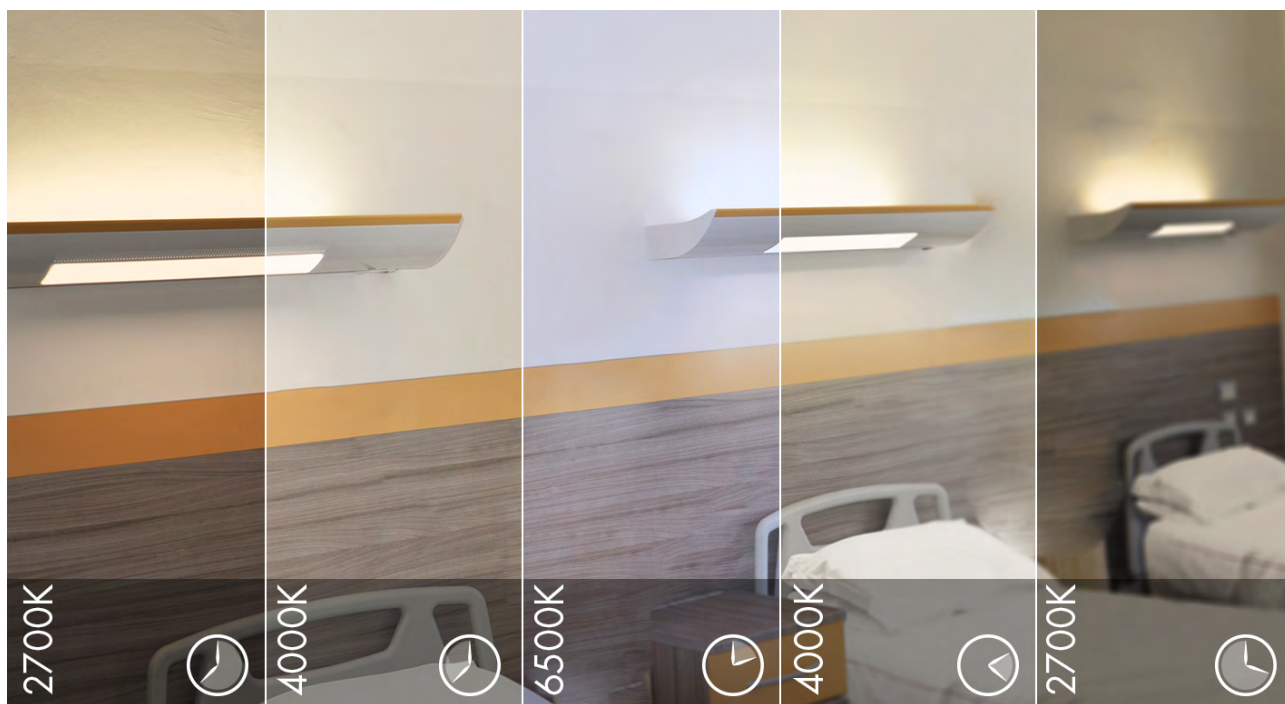
2700K

Dimensioni / **Dimensions** [mm]

L = 1070

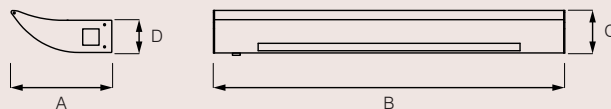
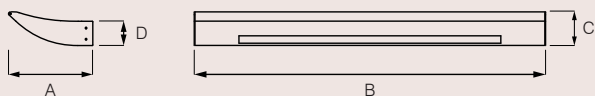
Cablaggio / **Wiring**

Driver Dimmerabile DALI TUNABLE
Dimmable DALI TUNABLE driver

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Steel structure, lateral aluminium end caps and aluminium front bar.

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*Glazed opal polycarbonate diffuser for indirect lighting.
Opal polycarbonate diffuser for direct lighting.*

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*Direct and indirect light emission bedhead unit with a simple and linear design that harmonizes with the styles and needs of nursing homes and rest homes. STAIR CARE is composed of two aluminum profiles designed for housing LED light sources for ambient light, reading light and, on request, night light. STAIR CARE can have a **custom finish in color**, this lamp is not just a bedhead unit but it can become the protagonist inside the room.*

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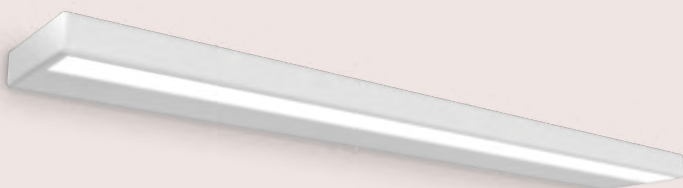
Direct and indirect light wall fixture characterized by a thin and minimal body.

*Its direct light, appropriately directed (wallwasher), ensures ideal lighting for healthcare personnel and at the same time optimal visual comfort patient. **SKYWALL** represents the right balance between design and technique for environments where discretion is of absolute importance.*

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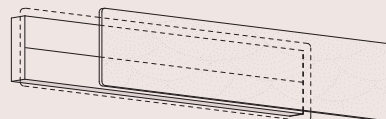
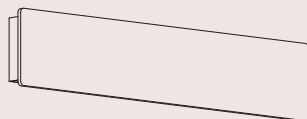
*Wall lamp with direct and indirect light, made of a double extruded aluminum profile, with reduced dimensions. Designed for nursing homes and rest homes patient rooms, it can also give character to the common areas dedicated to leisure and relaxation moments. For the patient room, thanks to the **MDF wooden accessory panel**, WAY WALL becomes not only a bedhead unit but also a decorative furnishing element that harmonizes easily with the environment.*

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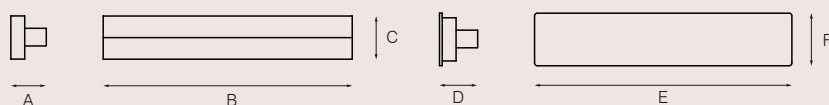
*LEVEL is a fine and elegant bedhead unit suitable for the installation in **rest houses** where no particular functions are requested. Its shape gives a perfect **direct lighting distribution**, obtained using the LED sources, avoiding the patient's glare and without disturbing other patients inside the room.*

LEVEL is a safety and modern device suitable to illuminate rooms with simplicity and efficiency, giving an optimal lighting level inside the room.

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*A simple wall lamp, with direct and indirect light emission, **with linear and square shapes**. Studied for nursing homes and rest homes patient room, it can also be applied in meeting and leisure areas, such as dining rooms and relaxing rooms. HORIZON gives a soft and comfortable light that makes the environment familiar and relaxing.*

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Opal polycarbonate diffuser
for direct light
Opal methacrylate diffuser
for indirect light

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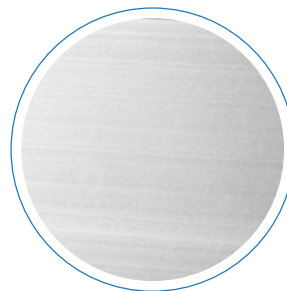
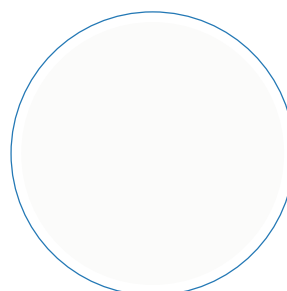
*ATENA LUX bedhead units can be anodized or painted. First of all, they are brushed and anodized in natural color: we talk about **anodizing** with reference to the process in which aluminium passes from its state of metal to the one of concrete surface by modifying the molecular structure, making its external surface definitely stronger.*

*Alternatively, bedhead units can be **painted** with polyester powders, UV rays stabilized and thermosetting at 180°C, with degreasing, phosphating and washing treatment.*

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*On request, it is possible to paint the devices with special paints with **anti-microbial effects**. This constantly protects the surfaces from bacteria, viruses, fungi and algae.*

*Differently from the common anti-bacterial paintings, this painting maintains its function unaltered across the years. It acts thanks to **metal ions** that are present in the powder paint film becoming active on the surface of the interested product, when a residual environmental humidity occurs. When the microbial organism comes to the surface, it is destroyed by the constant action of these long lasting ions, allowing the protection and the prophylaxis against parasites.*

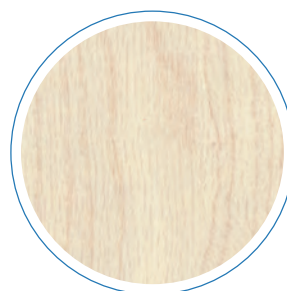
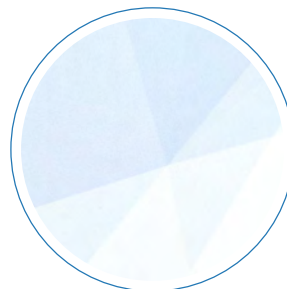
FINITURE DECORATIVE SPECIALI *SPECIAL DECORATIVE FINISHINGS*

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Thanks to different decoration techniques particular textures and images can be applied to the bedhead units.

*For some models ATENA LUX uses the **sublimation technique**: this process consists on transferring special colored inks from the sublimating film to the painting surface, impressing them under the action of an adequate temperature and pressure. The sublimation is realized with **eco-friendly substances** without Volatile Organic Compounds (VOC/VOC) as a guarantee for a **safe and harmless** graphic personalization.*

*For other models, through the process of "dipping" or "**water transfer printing**", it is possible to create three-dimensional decorations by immersion. As final treatment, a transparent painting is applied, with a glossy or matte finish, to protect the bedhead unit. Also this decorative technique has a **minimal environmental impact**: the manufacturing processes do not produce chemical residues subject to disposal and the water contained in the immersion tank can be reused for different dipping sessions.*

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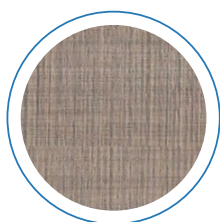
Our products ARMONIA, SANIMED and WAY WALL can be customized with the MDF wood panel.

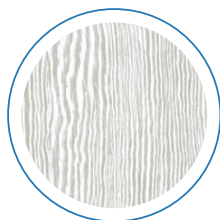
For these products it is possible to request a finish in line with the aesthetic and architectural taste of the rooms. Below we propose a choice of standard solutions, for specific needs other finishes are available on request.

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The following table shows a list of suggested available colors. All colours have a standard opaque finish, on request it can be glossy. For other colours, please contact the sales office.

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Atena Lux supplies the bedhead units complete with a call system keypad integrated. The optional accessories of Atena Lux bedhead units are defined during design stage and they can be modified according to the circumstances.

Modules with buttons and call system keypad can be furnished according to the specific call system required by the customer.

*The suggested teach has between **one and three commands** allowing the patient to call for nurse and service assistance (one command), together with the independent starting or turning off of the general and reading light (two and three commands). In models with more commands, the lighting control is provided by the installation of an electronic command in a lighting control relay.*

> *The teach has a 2-meter-long electric cable and a 7 poles spiral hook to guarantee a **quick disconnection**. When an accidental tear occurs, this avoids also the breaking of the 6 poles DIN socket connector located directly on the bedhead unit.*

> *The DIN hook integrates itself with the lighting fitting **without the use of an external plate**, maintaining the elegance of the product and allowing the homogeneity of elements.*

> *Thanks to the **ergonomic design** and to the **simplified handle**, the cleaning and the maintaining of the teach results easy.*

**SALE OPERATORIE
E ILLUMINAZIONE GENERALE**
***OPERATING THEATRES
AND GENERAL LIGHTING***

SISTEMA CERTIFICATO PER SALE OPERATORIE *CERTIFIED SYSTEM* *FOR OPERATING THEATRES*

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ATENA LUX has a complete range of specific lighting fittings for operating theatres and clean rooms.

*In collaboration with ATENA SPA, leader in Italy in the production of false metal ceilings, according to **specific project exigency** the company is able to provide a complete system that integrate lighting and false ceiling.*

*The lighting fittings of TAURUS family are developed to achieve a perfect technical and aesthetic **integration** between lighting and metal ceiling. These false ceilings are suitable for ceilings construction where ensuring stability between the plenum and the environment below is a need, assuring the maintenance of the environmental conditions required within the operating theatres and cleanrooms.*

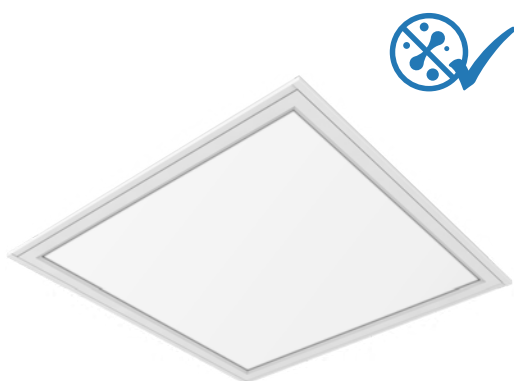
This system is certified for airtightness, resistance to chemical corrosion, micro-particle purity. The panels and lighting fittings, made of steel sheet and aluminum, are also treated with a special antimicrobial coating to help maintaining high hygiene standard in all sanitary rooms.

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TAURUS: LIGHTING FOR OPERATING THEATRES

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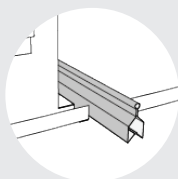
*Equipped with internal sealing gaskets, TAURUS guarantees **total protection** against dust and water jets (IP65), available with the following variants:*

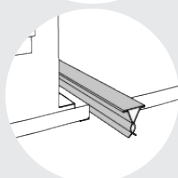
- › Dimensions: 300x1200 mm, 600x600 mm, 600x1200 mm
- › Diffuser: MOT (Opal methacrylate diffuser with high transmittance and uniformity)
- › Wiring: ON/OFF, DALI, EMERGENCY (1 or 3 hours of autonomy), RGB and HCL (on request)

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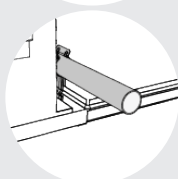
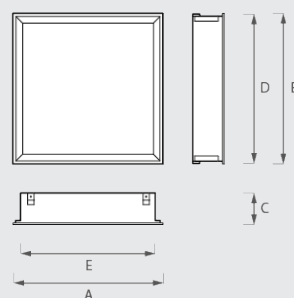
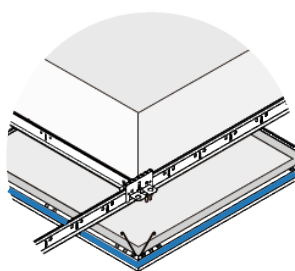
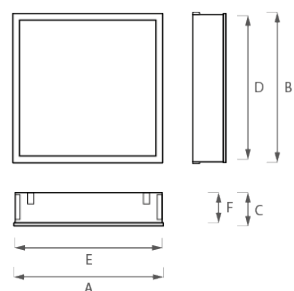
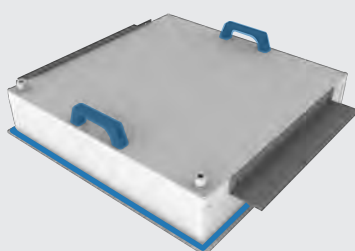
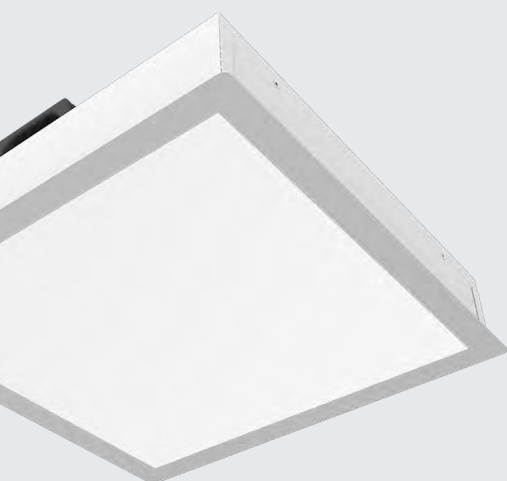
*Syncro Evo sealing system
consisting of sealing spring
and perimeter gasket*

Dimensioni / Dimensions [mm]					
A	B	C	D	E	F
600	600	135	592	591	118
600	1200	135	1192	591	118

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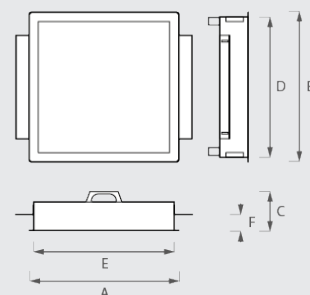
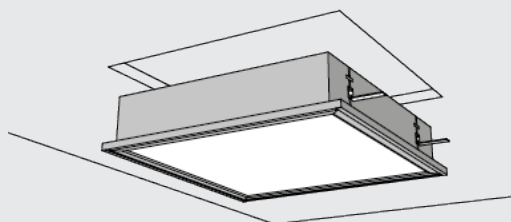
*Ergonomic handles
to facilitate installation from above and
sealing gasket*

Dimensioni / Dimensions [mm]					
A	B	C	D	E	F
650	650	170	612	612	30 ÷ 90
350	1250	170	312	1212	30 ÷ 90
650	1250	170	612	1212	30 ÷ 90

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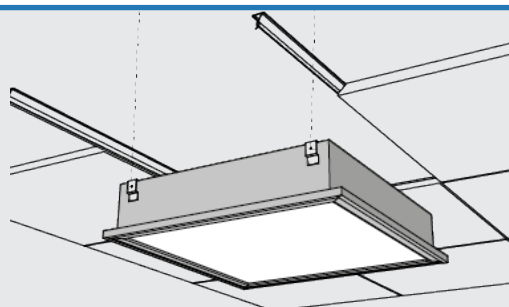
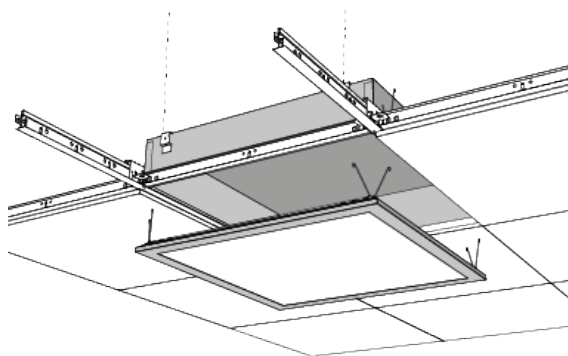
Hook-down modular panels false ceiling with clip-in structure (triangle profile or "Elios" tubular profiles, 16 and 22mm diameter and slinging springs).

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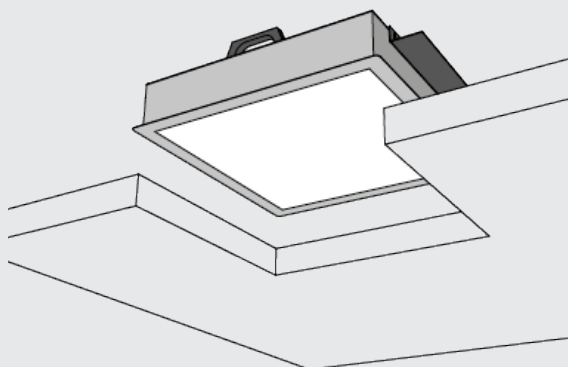
Atena Syncro evo® metal tiles false ceiling for T24 grid, properly conceived for hospitals. The special springs, hooked on T grid, make an high pressure on neoprene gasket installed along the tile, guaranteeing the enviroment cleaning.

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Walkable false ceiling, inspectable from above, in modular panels (sandwich or monopanel type).

The product frame allows an installation in perfect continuity with the false ceiling, mounting kit with adjustable fixing brackets supplied.

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Besides bedhead units for patient rooms, intensive care and laboratories, ATENA LUX realizes also lighting fittings for the other places of a healthcare building. We illustrate below some of our suggestions for general places and we invite You to consult our web site www.atenalux.com for more information.

● SALE OPERATORIE, PREPARAZIONE E RISVEGLIO *OPERATING, PREPARATION AND RECOVERY ROOMS*

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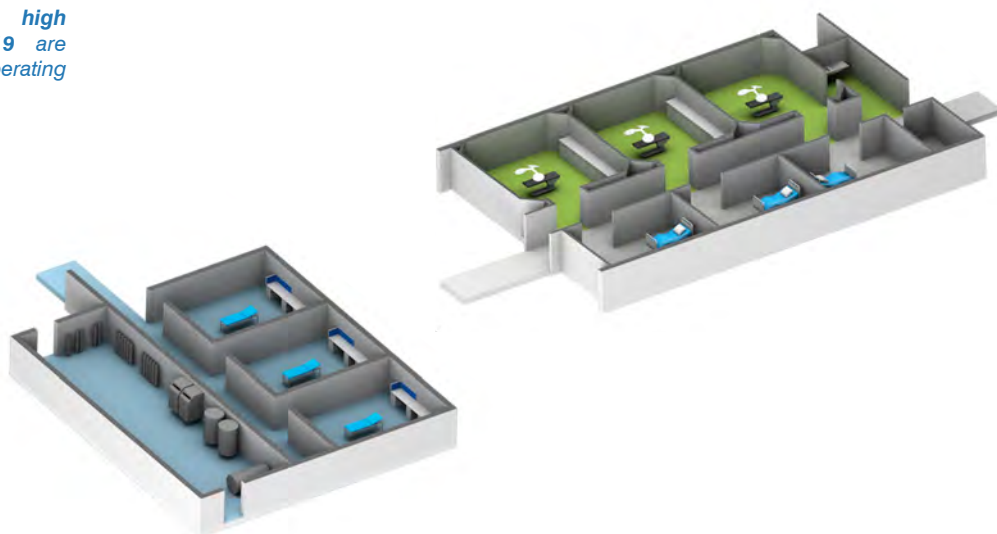
*White light with **tunable white**, **high protection grade** and **UGR<19** are necessary characteristics in operating theatres and awakening rooms.*

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*In places like warehouses, laundry service, technical places and autopsy rooms the requested protection grade is **IP65** or **IP66**, as a guarantee of total protection against dust and liquid substances.*

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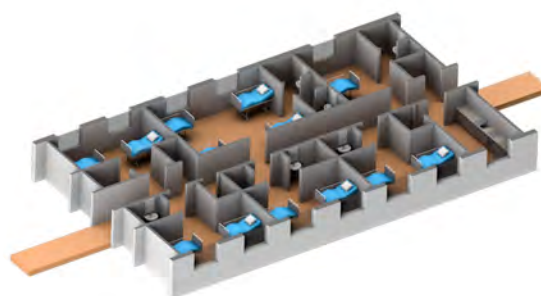
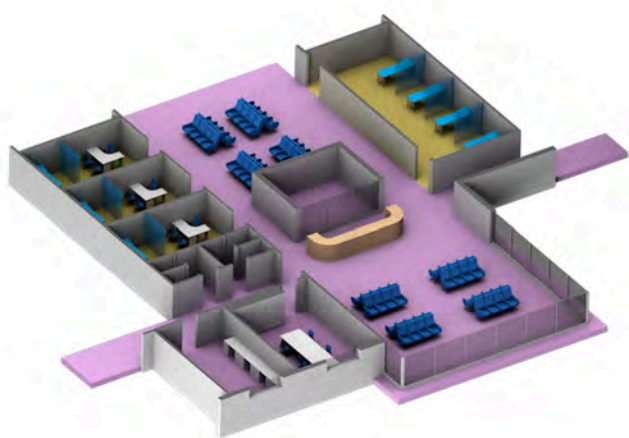
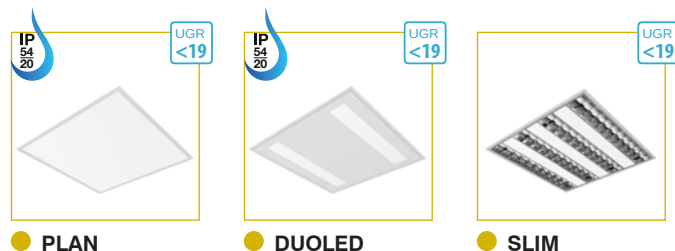
*A **correct lighting** helps the medical staff doing its activities in the best way during the day and, at the same time, it improves the patient's relaxation during the examination.*

● UFFICI E RECEPTION OFFICES AND RECEPTION

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*The offices of the medical structures are illuminated with lighting fittings suitable for the installation with **visual display units** while general areas such as reception and waiting rooms are enlightened with **decorative lighting fittings** with a particular design, in order to give familiarity and style to the spaces.*

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*It is possible to install some **lighting fittings for ceiling or wall** in the patient room, elements able to give particular protection grades and lighting functions.*

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